



SmileHaus Instruction Manual — Teeth Whitening Range

Teeth Whitening Pen (4ml)

Product Overview

The Teeth Whitening Pen is a convenient, targeted whitening solution designed for cosmetic teeth whitening. It allows controlled application of whitening gel directly to the enamel surface and is suitable for salon use, mobile treatments, and at-home whitening routines.

This product contains an Australian-compliant whitening gel registered with the **National Industrial Chemicals Notification and Assessment Scheme (NICNAS)**.

Gel Ingredients

Glycerin, Propylene Glycol, Deionized Water, Hydrogen Peroxide (6%), Carbomer, EDTA, PVP, Menthol.

How to Use

1. Brush and dry teeth thoroughly before application.
2. Twist the base of the pen to dispense a small amount of gel.
3. Apply a thin, even layer directly onto the front surfaces of the teeth.
4. Avoid contact with gums and lips.
5. Leave the gel on teeth for **15 minutes**.
6. Rinse mouth thoroughly after use.

Best results are achieved when used **multiple times per week** as part of a consistent whitening routine.

Storage & Care

- Store whitening pens in a **refrigerator upon receipt** to ensure gel longevity and activation.
- When refrigerated, pens can last **9–12 months**.
- In cooler climates, pens may be stored in a cool, dark place.
- In hot climates, refrigeration is recommended at all times.
- Remove the pen from the fridge **a few hours prior to use** to allow the gel to reach room temperature.

Note: Gel consistency may vary between pens; this does not affect whitening performance or results.

Safety Information

- Do not swallow gel.
- Avoid contact with soft tissue.
- Discontinue use if irritation or sensitivity occurs.
- Keep out of reach of children.

Australian Compliance & Cosmetic Use Disclaimer

This product is intended for **cosmetic use only** and is not a therapeutic or dental treatment under Australian regulations. It is not designed to diagnose, treat, cure, or prevent any disease or oral health condition. Results may vary between individuals. If you have existing dental conditions, restorations, gum disease, or experience prolonged sensitivity, consult a registered dental professional prior to use. Use only as directed and in accordance with Australian Consumer Law and cosmetic product safety standards.



Thank you,

Best regards,
SmileHaus Aus

SmileHausAus@outlook.com